

Ask A-dec® about...



WATERLINE SHOCK TREATMENT

Fast-acting liquid shock solution removes odor and foul-tasting bacteria in dental unit waterlines.

Just mix the two solutions together into an empty water bottle, run through the waterlines, and soak overnight. The blue color lets you know that the solution is in the waterlines. After completing the shock treatment, flush the waterlines with water and resume your regular use.



- Penetrates and removes the build-up of odor and foul-tasting microbial contamination bacteria.
- Will not corrode or clog waterlines.
- Blue color lets you know the solution is hard at work in the waterlines.
- Non-corrosive and gentle on equipment.
- Odorless formula.
- Recommended by A-dec, for A-dec equipment.





When used in conjunction with regular testing and adherence to protocols, A-dec ICX and ICX Renew help simplify your waterline maintenance routine.

Product description	Applications	A-dec part number
ICX Renew - Case	36	90.1691.00
ICX Renew - Box	9	90.1691.01



Unwanted guests in your waterlines?



Turn off delivery system, remove and empty the water bottle.

Remove syringe detachable mo ultrasonic hand



Remove syringe tips, handpieces, detachable motors and couplings, and ultrasonic handpieces from their tubing.



Add the contents of both ICX Renew bottles (solutions 1 & 2) and swirl to let the product coat all surfaces.




Re-attach water bottle and turn delivery system on.



Run solution through the Dental Unit Waterlines (DUWL) until you start to see blue water from the tubing and cuspidor cupfill (where relevant).



Let the product sit in the DUWL for 12-96 hours.

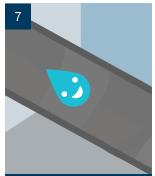

Turn on delivery system and flush remaining solution through DUWL.



Rinse water bottle thoroughly. Refill bottle with water and flush through DUWL until empty.



Add ICX tablet to new water to maintain your Dental Unit Waterlines.



When you SHOCK, let ICX RENEW.



Find out more about ICX Renew Products.

a-dec.com/icx



